

Electrosurgical Capacitive Current Gel Pad

The OKLand Reusable Patient Return Electrode Pad combines innovative patient return electrode technology with a safe and reusable pressure reducing pad. It removes the need for disposable sticky plates during electrosurgical procedures. The pad is filled with a viscoelastic polymer, which effectively reduces patient injury associated with direct pressure, shearing and friction. The OKLand PRE Pad is ideal for a wide variety of surgical procedures, making it a key item for both safety & simplicity during surgery.



Safety

- Safer than sticky pads
- No adhesive-related skin irritations or tears
- Bio-compatible viscoelastic gel pad reduces risk of pressure injury
- Can be used without concern for scars, burns, tattoos or metal implants
- Universal design for all sizes, suitable for infants weighing >350g, with no upper weight limit.
- The current-limiting technology makes the safety advantages exceptional.



Efficiency

- No shaving or searching for adequate patient pad site
- Compatible with all ESU's
- Can be used with any operating table and patient position
- Effective even in presence of clothing, hospital gown or sheet between patient and pad
- The Dual Cord Pad has two electro-surgical connectors, enabling two ESU generators to be used at the same time. This considerably improves ease for staff, as it ends the need for multiple sticky plate placement sites




Sustainability

- Eliminates waste of disposable pads

Value

- Significantly reduces costs over 24 months lifecycle, compared to sticky pads



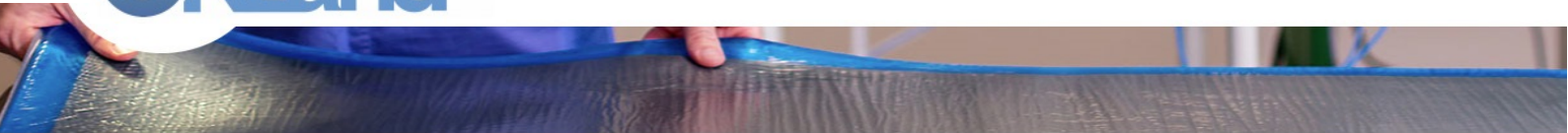

Electrosurgical Capacitive Current Gel Pad

Suitable for all patient positions



Supine Positioning Surgery

Lateral Positioning Surgery

Lithotomy Positioning Surgery

Prostrate Positioning Surgery

Instructions for Use

- Place the OKLand Reusable Patient Return Electrode directly on the Operating Room table
- Position patient directly onto the OKLand Reusable Patient Electrode Pad with the patient's torso extending half the length of the pad
- The OKLand Reusable Patient Return Electrode can be used with a variety of surgical procedures
- Patient weight limitations > 0.35 kg
- Cleaning Instructions < 80% alcohol solution / wipe

Ordering Information

Product Number	Description	Dimensions
OKAHD-001	PRE Pad - Single Cable	120 x 50 x 1.3cm
OKAHD-002	PRE Pad - Single Cable - Paediatrics	41 x 41 x 1.3cm
OKAHD-003	PRE Pad - Single Cable - Paediatrics	85 x 40 x 1.3cm
OKAHD-004	PRE Pad - Dual Cable	120 x 50 x 1.3cm
OKAHD-005	PRE Pad - Single Cable	100 x 50 x 1.3cm
OKAHD-006-100	PRE Pad - Single Cable	100 x 50 x 0.3cm
OKAHD-006-120	PRE Pad - Single Cable	120 x 50 x 0.3cm
OKAHD-007	PRE Pad - Dual Cable	120 x 50 x 0.3cm

ESU Compatible Cable included