



OKLand Patient Reusable Return Electrode Pad

Ensure Patient Safety

The OKLand Patient Reusable Return Electrode Pad combines the innovative patient return electrode technology:

- The patient lies on the pad to achieve full and safe electrical patient contact.
- Safety still prevails even when the patient is in contact with only a small portion of the pad.
- The OKLand Pad is a self-contained current limiting device.

Safety

- Safer than sticky-style return electrode pads.
- Large electrode size and capacitive 'gatekeeper' design ensures patient safety.

Saves Time

- No shaving or searching for an adequate patient pad site.

Minimises Surgical Pressure Injuries

- No adhesive-related skin irritations and tears with all the benefits of pressure reduction pad.

Saves Money

- The reusable patient return electrode pad significantly reduces cost per procedure.

Environmentally Friendly

- Eliminates the waste of disposable sticky pads.

Allergy Friendly

- Latex Free.

Instructions for Use


- Place the OKLand Reusable Patient Return Electrode Pad directly on to the Operating Theatre table.
- Position patient directly on to the Reusable Patient Electrode Pad with the adult's torso extending half the length of the pad.
- Patient weight limitations >0.35kg
- Cleaning Instructions <80% alcohol solution/wipe




OKLand Patient Reusable Return Electrode Pad




Illustrations of Surgery Positioning Guide

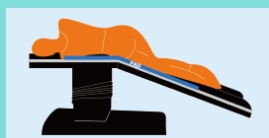

Lateral Positioning Surgery

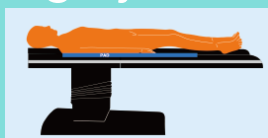

Supine Positioning Surgery

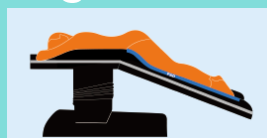









Prostrate Positioning Surgery



Lithotomy Positioning Surgery

Ordering Information

Cabling included with each purchase

Catalogue Number	Description	Size	Case Qty	Catalogue Number	Description	Size	Case Qty
OKAHD-001	 Patient Return Electrode Pad Single Cable, min patient weight 0.35kg	120 x 50 x 1.3cm	1	OKAHD-005	 Patient Return Electrode Pad Single Cable, min patient weight 0.35kg	100 x 50 x 1.3cm	1
OKAHD-002	 Patient Return Electrode Pad Single Cable, min patient weight 0.35kg	41 x 41 x 1.3cm	1	OKAHD-006-100	 Patient Return Electrode Pad Single Cable, min patient weight 0.35kg	100 x 50 x 0.3cm	1
OKAHD-003	 Patient Return Electrode Pad Single Cable Pediatric, min patient weight 0.35kg	85 x 40 x 1.3cm	1	OKAHD-006-120	 Patient Return Electrode Pad Single Cable, min patient weight 0.35kg	120 x 50 x 0.3cm	1
OKAHD-004	 Patient Return Electrode Pad Dual Cable, min patient weight 0.35kg	120 x 50 x 1.3cm	1	OKAHD-007	 Patient Return Electrode Pad Dual Cable, min patient weight 0.35kg	120 x 50 x 0.3cm	1
Catalogue Number	Description	Case Qty	Catalogue Number	Description	Case Qty		
OKCC-01	Connecting Cable (Square Head, Single)	1	OKCC-05	Connecting Cable (Flat Head, Single)	1		
OKCC-02	Connecting Cable (Square Head, Dual)	1	OKCC-06	Connecting Cable (Flat Head, Dual)	1		
OKCC-03	Connecting Cable (Mic Head, Single)	1	OKCC-07	Connecting Cable (Ext. Square, Single)	1		
OKCC-04	Connecting Cable (Mic Head, Dual)	1	OKCC-08	Connecting Cable (Ext. Square, Dual)	1		